



Pessary – Cube

Datasheets





Descriptions

The **Cube Pessary** is a flexible, cube-shaped vaginal device made of medical-grade silicone. It is designed with concave surface on each side to create suction, allowing it to stay in place even in cases of severe pelvic organ prolapse (POP). Unlike other pessaries that rely on support from vaginal walls, the cube pessary uses suction to adhere to the vaginal walls, providing stronger support for advanced prolapse cases. It must be removed and cleaned daily due to the way it adheres to vaginal tissue.

Specifications

Part Number	Fitting Set	#0		#1		#2		#3		#4		#5		Feature
		Inch	mm	Inch	mm	Inch	mm	Inch	mm	Inch	mm	Inch	mm	
PCU	Available	1.00	25	1.15	29	1.30	33	1.45	37	1.60	41	1.75	45	
PCUD	None	1.00	25	1.15	29	1.30	33	1.45	37	1.60	41	1.75	45	With Drain
Part Number	Fitting Set	#6		#7		#8		#9		#10		Feature		
		Inch	mm	Inch	mm	Inch	mm	Inch	mm	Inch	mm			
PCU	None	1.95	50	2.20	56	2.50	63	2.75	70	2.95	75			
PCUD	Available	1.95	50	2.20	56	2.50	63	2.75	70	2.95	75	With Drain		

Packing

Part Number	Dimensions (mm)			Quantity			Weight / Carton (kg)	
	Pouch	Box	Carton	Q'ty / Box	Box / Carton	Q'ty / Carton	Net	Gross
PCU00B1	120x316x26	180x130x165	369x352x411	10	12	120	1.38	5.16
PCU01B1	120x316x30	180x130x165	369x352x411	10	12	120	1.68	5.46
PCU02B1	120x316x34	180x130x165	369x352x411	20	6	120	2.10	5.65
PCU03B1	120x316x38	180x130x165	369x352x411	20	6	120	2.88	6.43
PCU04B1	120x316x42	180x130x165	369x352x411	20	6	120	3.60	7.15
PCU05B1	120x316x46	180x130x165	369x352x411	20	6	120	4.56	8.11
PCU06B1	120x356x51	180x130x165	369x352x411	10	6	60	3.00	5.60
PCU07B1	120x356x57	180x130x165	369x352x411	10	6	60	3.87	6.47
PCU08B1	120x356x64	180x130x165	369x352x411	5	6	30	3.00	4.99
PCU09B1	120x356x71	180x130x165	369x352x411	5	6	30	3.63	5.62
PCU10B1	120x356x76	180x130x165	369x352x411	5	6	30	4.38	6.37
PCUD00B1	120x316x26	180x130x165	369x352x411	10	12	120	1.26	5.04
PCUD01B1	120x316x30	180x130x165	369x352x411	10	12	120	1.44	5.22
PCUD02B1	120x316x34	180x130x165	369x352x411	20	6	120	1.74	5.29
PCUD03B1	120x316x38	180x130x165	369x352x411	20	6	120	2.76	6.31
PCUD04B1	120x316x42	180x130x165	369x352x411	20	6	120	3.48	7.03
PCUD05B1	120x316x46	180x130x165	369x352x411	20	6	120	4.44	7.99
PCUD06B1	120x356x51	180x130x165	369x352x411	10	6	60	2.94	5.54
PCUD07B1	120x356x57	180x130x165	369x352x411	10	6	60	3.81	6.41
PCUD08B1	120x356x64	180x130x165	369x352x411	5	6	30	2.94	4.93
PCUD09B1	120x356x71	180x130x165	369x352x411	5	6	30	3.57	5.56
PCUD10B1	120x356x76	180x130x165	369x352x411	5	6	30	4.29	6.28

Applications

The Cube Pessary is used in the management of:

- Severe Pelvic Organ Prolapse (POP)** : Effective for **grade III and IV prolapse**, including **uterine prolapse, cystocele (bladder prolapse), and rectocele (rectal prolapse)**.
- Patients with Weak Vaginal Tone** : Beneficial for women with significant **vaginal laxity**, where other pessary types may not stay in place.
- Alternative to Surgery** : Suitable for women who cannot or do not want to undergo surgery for prolapse.

Precautions

- Daily Removal is Necessary** : The cube pessary **must be removed, cleaned, and reinserted every day** to prevent complications such as infection, tissue damage, or vaginal ulcers.
- Proper Fitting is Essential** : A healthcare provider should determine the correct size for comfort and effectiveness.
- Risk of Vaginal Irritation and Ulcers** : Due to the suction effect, prolonged use without removal can lead to **tissue erosion, ulcer formation, and infection**.
- Monitor for Increased Vaginal Discharge or Bleeding** : These may indicate irritation, infection, or improper fit.
- Requires Patient Dexterity or Assistance** : Since it needs **daily removal**, women who have difficulty handling the pessary may require caregiver assistance or reconsider alternative pessary types.
- Not Ideal for Urinary Incontinence Management** : Unlike a ring or dish pessary with a knob, the **cube pessary does not provide significant urethral support** for stress urinary incontinence (SUI).

FDA 510(k) K031463



LI Medical Corporation

2F, No. 43, Zhongxing Road, Xizhi District,
New Taipei City 221012, Taiwan
Tel : +886-2-8646-1999
Fax : +886-2-2647-1999

Representative / Distributor