




Cervical Dilator (Sterile)

Datasheets

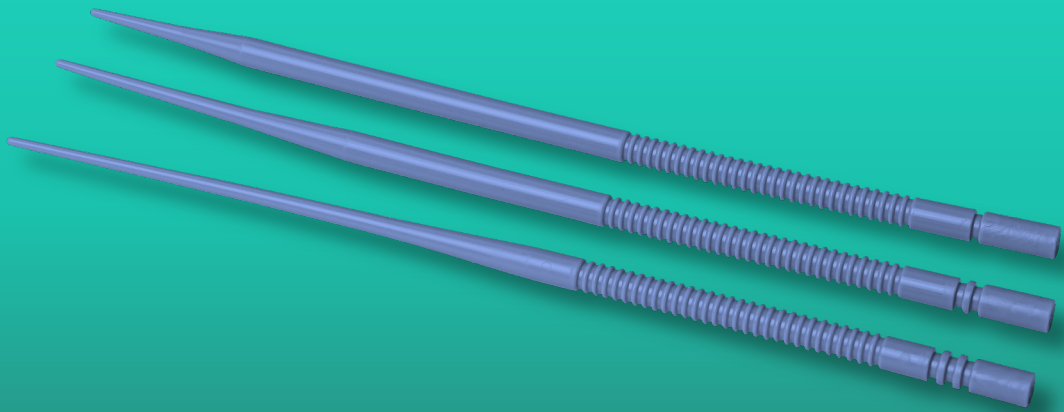



Applications

A cervical dilator is a medical instrument used to gradually dilate the cervix, the opening of the uterus. This procedure, known as cervical dilation, is often performed before certain medical procedures, such as hysteroscopy or endometrial biopsy.

Specifications

| Part Number | Length (mm) | Tip Length (mm) | Handle Diameter (mm) | Color | Feature |
|-------------|-------------|-----------------|----------------------|-------|----------------|
| DC01B1 | 210 | 38 | 6.4 | Blue | OS Finder |
| DC02B1 | 210 | 69 | 6.4 | Blue | Canal Locator |
| DC03B1 | 210 | 115 | 6.4 | Blue | Fundus Dilator |

Packing

| Part Number | Dimensions (mm) | | | Quantity | | | Weight / Carton (kg) | |
|-------------|-----------------|---------------|-----------------|------------|------------|-------------|----------------------|-------|
| | Pouch | Box | Carton | Q'ty / Box | Box/Carton | Q'ty/Carton | Net | Gross |
| DC01B1 | 60 x 285 x 8 | 75 x 50 x 283 | 369 x 352 x 411 | 25 | 40 | 1,000 | 5.20 | 9.63 |
| DC02B1 | 60 x 285 x 8 | 75 x 50 x 283 | 369 x 352 x 411 | 25 | 40 | 1,000 | 4.40 | 8.83 |
| DC03B1 | 60 x 285 x 8 | 75 x 50 x 283 | 369 x 352 x 411 | 25 | 40 | 1,000 | 3.40 | 7.83 |

Instructions for Use

Specific instructions may vary depending on the type of dilator and the procedure being performed. Always follow the manufacturer's instructions and adhere to aseptic techniques.

01. Sterilize the dilators and other necessary instruments.
02. Prepare the patient for the procedure, ensuring they are comfortable and relaxed.
03. Visually inspect the cervix to assess its size and any abnormalities.
04. Use a speculum to expose the cervix.
05. Start with the smallest dilator and gradually progress to larger sizes.
06. Insert the dilator slowly and gently, rotating it slightly as you advance.
07. Apply gentle pressure to dilate the cervix.
08. Monitor the patient for any signs of discomfort or pain.
09. Remove the dilators carefully.
10. Monitor the patient for any bleeding or cramping.
11. Provide pain relief as needed.

Precautions

01. Maintain strict aseptic technique throughout the procedure to prevent infection.
02. Insert the dilators slowly and gently to avoid injury to the cervical tissue.
03. Use local anesthesia or other pain management techniques to minimize discomfort.
04. Monitor the patient for any signs of complications, such as excessive bleeding or uterine perforation.
05. Dispose of used dilators according to medical waste disposal guidelines.

It's important to note that cervical dilation is a medical procedure that should be performed by a qualified healthcare professional. Self-dilation is not recommended and can lead to serious complications.

FDA Listed



LI Medical Corporation

2F, No. 43, Zhongxing Road, Xizhi District,
New Taipei City 221012, Taiwan
Tel : +886-2-8646-1999
Fax : +886-2-2647-1999

Representative / Distributor