



Pessary — MarLand

Datasheets





Descriptions

The **MarLand Pessary** is a specialized vaginal pessary designed to provide **enhanced support** for pelvic organ prolapse (POP) and associated conditions. It has a unique shape, combining features of a **ring pessary** and a **dish pessary**, providing **stronger support** than a standard ring pessary while remaining relatively easy to insert and remove. It is made of **medical-grade silicone**, which is flexible, durable, and comfortable for long-term use.

Specifications

Part Number	Fitting Set	#0		#1		#2		#3		#4		#5		Feature
		Inch	mm	Inch	mm	Inch	mm	Inch	mm	Inch	mm	Inch	mm	
PMA	None	-	-	-	-	2.25	57	2.50	64	2.75	70	3.00	76	
PMAS	Available	-	-	-	-	2.25	57	2.50	64	2.75	70	3.00	76	With Support
Part Number	Fitting Set	#6		#7		#8		#9		#10				Feature
		Inch	mm	Inch	mm	Inch	mm	Inch	mm	Inch	mm	Inch	mm	
PMA	None	3.25	83	3.50	89	3.75	95	-	-	-	-			
PMAS	Available	3.25	83	3.50	89	3.75	95	-	-	-	-			With Support

Packing

Part Number	Dimensions (mm)			Quantity			Weight / Carton (kg)	
	Pouch	Box	Carton	Q'ty / Box	Box / Carton	Q'ty / Carton	Net	Gross
PMA02B1	120x316x35	180x130x165	369x352x411	10	12	120	4.32	8.10
PMA03B1	120x316x37	180x130x165	369x352x411	10	12	120	4.44	8.22
PMA04B1	120x316x39	360x130x165	369x352x411	10	12	120	5.10	8.88
PMA05B1	120x316x41	360x130x165	369x352x411	5	12	60	2.70	5.40
PMA06B1	120x356x43	360x130x165	369x352x411	5	12	60	2.94	5.77
PMA07B1	120x356x45	360x130x165	369x352x411	5	12	60	3.30	6.13
PMA08B1	120x356x47	360x130x165	369x352x411	5	12	60	3.54	6.37
PMAS02B1	120x316x35	180x130x165	369x352x411	10	12	120	4.32	8.10
PMAS03B1	120x316x37	180x130x165	369x352x411	10	12	120	4.44	8.22
PMAS04B1	120x316x39	360x130x165	369x352x411	10	12	120	5.10	8.88
PMAS05B1	120x316x41	360x130x165	369x352x411	5	12	60	2.70	5.40
PMAS06B1	120x356x43	360x130x165	369x352x411	5	12	60	2.94	5.77
PMAS07B1	120x356x45	360x130x165	369x352x411	5	12	60	3.30	6.13
PMAS08B1	120x356x47	360x130x165	369x352x411	5	12	60	3.54	6.37

Applications

The MarLand pessary is used in the management of:

- 1. Pelvic Organ Prolapse (POP)** : Provides support for **moderate to severe uterine prolapse**, cystocele (bladder prolapse), and rectocele (rectal prolapse).
- 2. Stress Urinary Incontinence (SUI)** : Helps control urinary leakage by supporting the urethra.
- 3. Post-Surgical Support** : Used to maintain pelvic support and prevent recurrence of prolapse after surgery.
- 4. Alternative to Surgery** : Suitable for patients who cannot undergo surgical correction of prolapse due to age, health conditions, or personal preference.

Precautions

- 1. Proper Fitting is Crucial** : A healthcare provider must determine the correct size to ensure comfort and effectiveness.
- 2. Regular Follow-ups** : Patients should have regular check-ups to assess the pessary's condition and vaginal health, typically every few months.
- 3. Cleaning and Maintenance** : The pessary should be removed, cleaned with mild soap and water, and reinserted per the doctor's instructions (usually weekly or as advised).
- 4. Watch for Symptoms of Irritation or Infection** : Increased vaginal discharge, foul odor, bleeding, or discomfort could indicate an infection, ulceration, or incorrect fit.
- 5. Avoid Prolonged Use Without Monitoring** : Leaving the pessary in place for extended periods without removal or medical check-ups can lead to **vaginal ulceration, infections, or tissue damage**.
- 6. Monitor for Allergic Reactions** : While rare, some individuals may experience sensitivities to the materials used in the pessary.

FDA 510(k) K031463



LI Medical Corporation

2F, No. 43, Zhongxing Road, Xizhi District,
New Taipei City 221012, Taiwan
Tel : +886-2-8646-1999
Fax : +886-2-2647-1999

Representative / Distributor